Diasorin

LIAISON® Q.S.E.T.

Quick Pick Kit

EN: For more information,
scan the code

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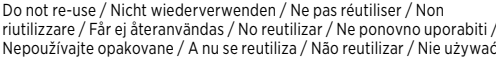
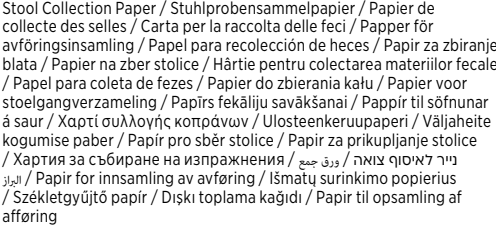
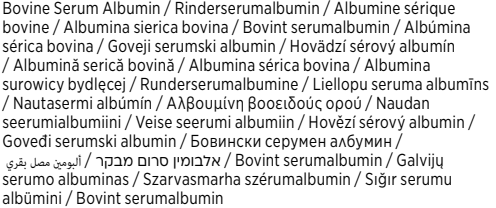
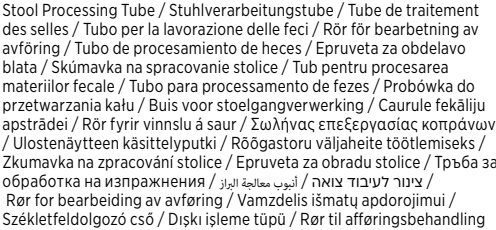
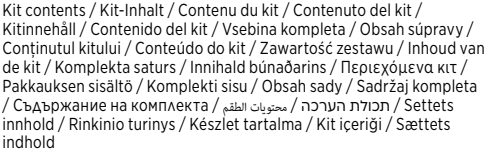
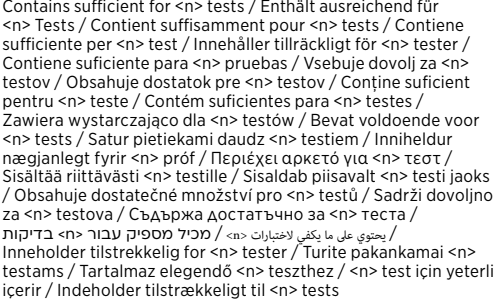
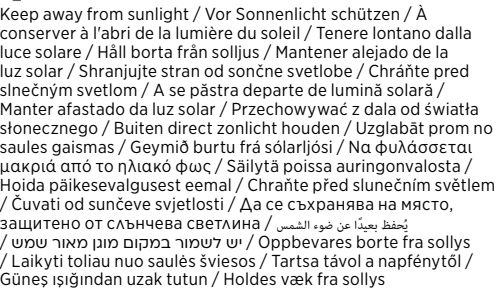
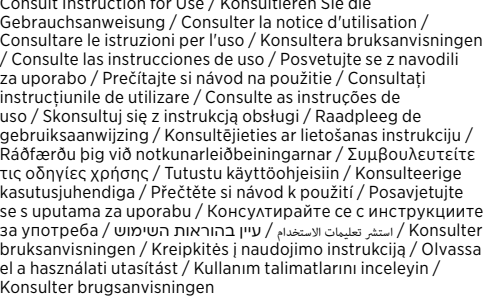
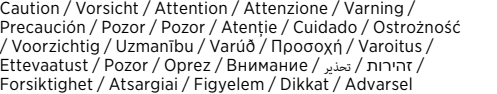
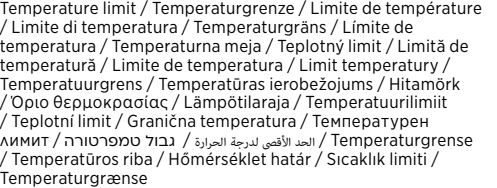
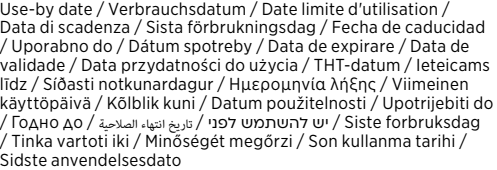
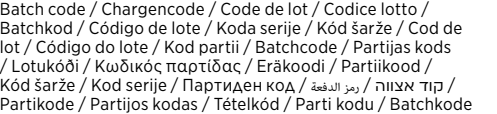
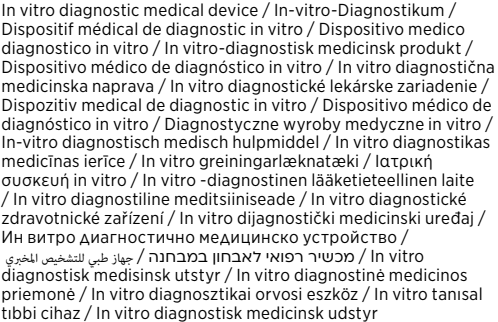
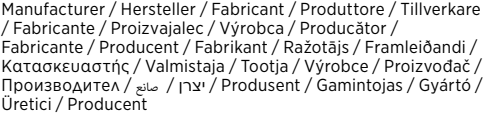
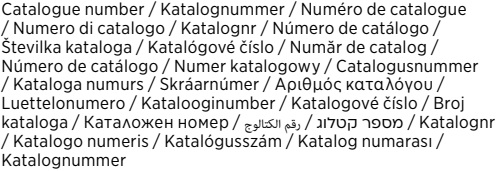
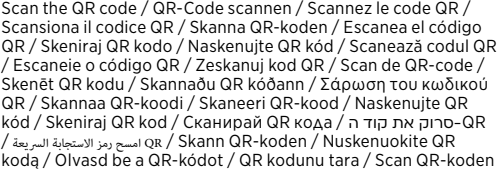
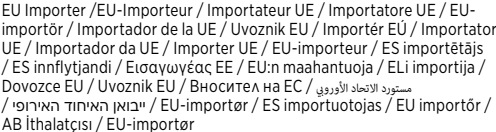
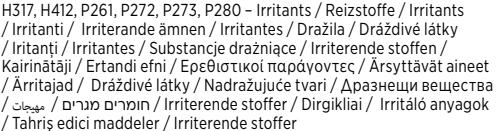
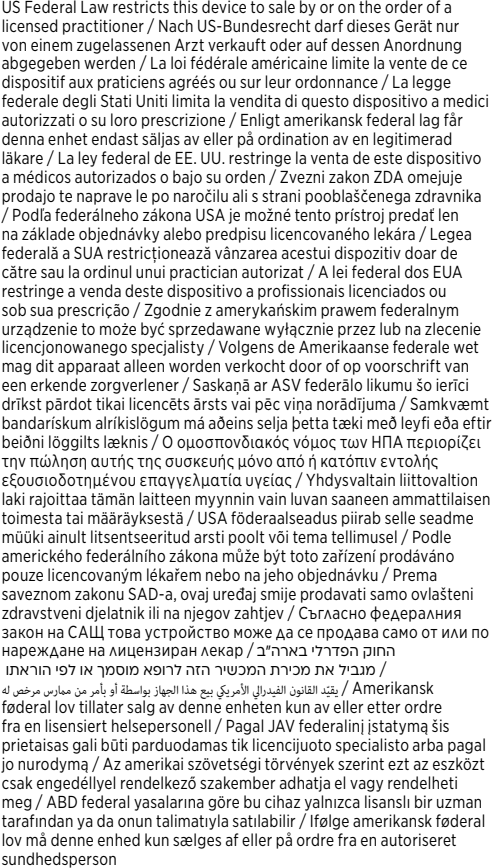
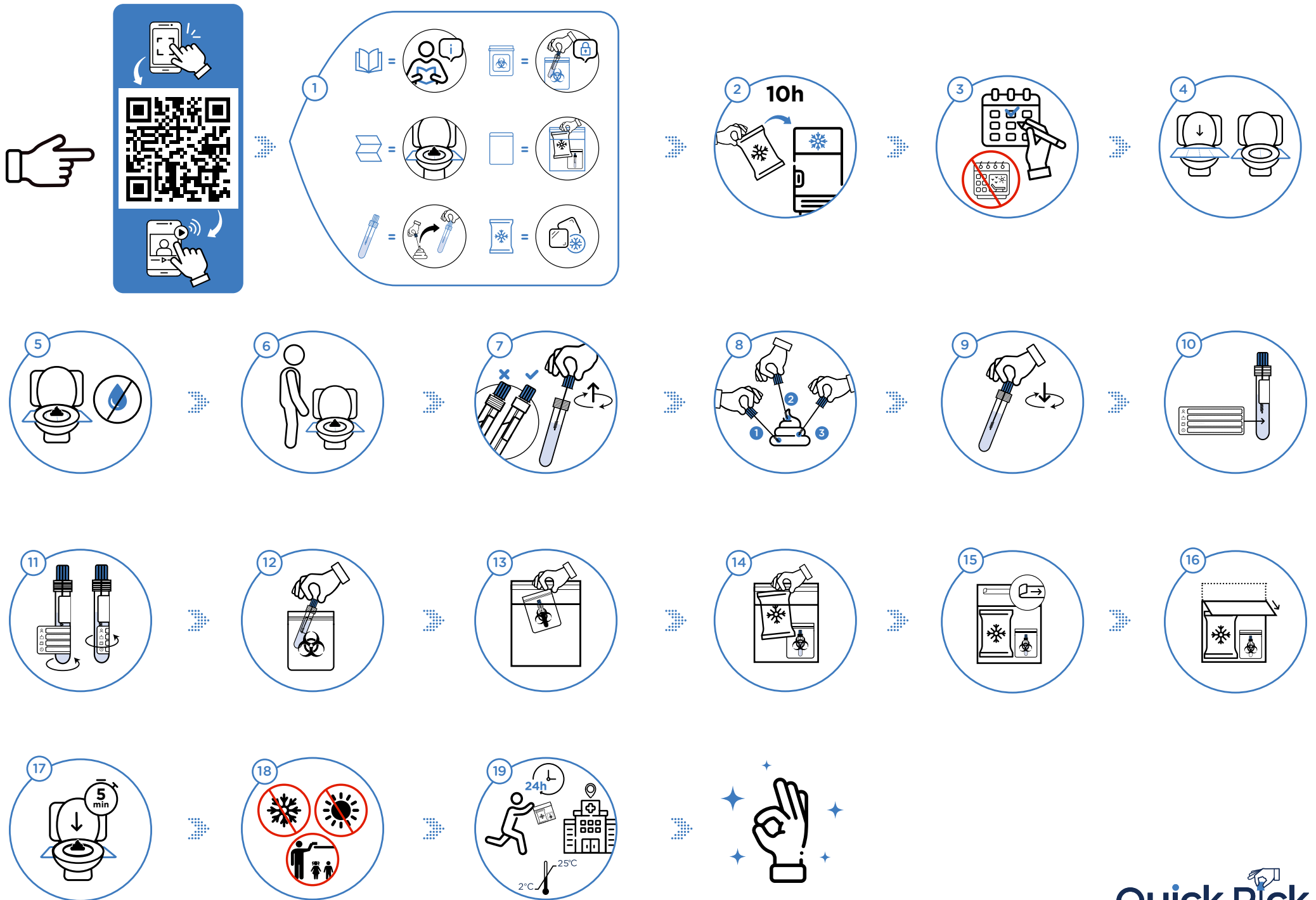
Read this instruction carefully and follow these steps to collect your stool. Children under 14 must be assisted by an adult for stool collection. For more infos scan the QR code or visit www.qset-quickpick.com to watch a short demo video.

1. Verify all components before sample collection. If any are missing, contact your healthcare provider.
2. Place only the provided ice pack in the freezer for at least 10 hours before sample collection.
3. Return sample within 24h of collection. Don't collect if you can't meet this timeframe.
4. Use adhesive strips to place stool paper under the seat and over the toilet bowl.
5. Sit on the toilet and collect stool on the paper. Avoid urine. If stool is liquid or bloody, contact your healthcare provider.
6. Once the bowel movement is complete and stool is on the collection paper, stand up and collect the sample using the LIAISON® Q.S.E.T. Quick Pick.
7. Unscrew the blue cap of the LIAISON® Q.S.E.T. Quick Pick and remove the sample collection wand. Do NOT unscrew the clear cap.
8. While holding the blue cap, dip the sample collection wand into the stool sample at 3-5 sites. Ensure the grooves of the wand are fully coated with the stool sample.
9. Insert the wand back through the opening of the tube and tightly screw into place. Excess stool will be removed from the wand by the inner funnel.
10. Label your sample clearly using block letters. Proper labeling is essential for identification. Instructions and label are on the last page.
11. Wrap the label around the bottom of the tube.
12. Put the LIAISON® Q.S.E.T.Quick Pick in the biohazard bag with absorbent pad. Ensure the bag is tightly sealed.
13. Place the biohazard bag containing the LIAISON® Q.S.E.T. Quick Pick inside the insulated pouch.
14. Place the frozen ice pack inside the insulated pouch.
15. Remove the adhesive strip from the insulated pouch.
16. Seal the insulated pouch using the adhesive strip.
17. The collection paper may now be carefully removed from the toilet bowl. Throw it into the toilet and wait 5 minutes. Flush down the eco-friendly biodegradable collection paper.
18. Store sample at 2-25 °C, away from heat/sunlight. Do not freeze. Keep out of reach of children.
19. The collection site must receive your sample within 24 hours from the time of collection. Stool collection completed! You'll receive your test results from your healthcare provider.

WARNINGS

Do **NOT** drink the liquid inside the LIAISON® Q.S.E.T. Quick Pick device (the sample tube).

Do **NOT** let the liquid touch your skin or your eyes. If the liquid comes in contact with your skin or eyes, rinse thoroughly with water. Use of gloves is recommended.

Do **NOT** insert the LIAISON® Q.S.E.T. Quick Pick device or any part of it into your body.

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**DiaSorin Inc.**